SINO EURO LEAD

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<https://www.sino-euro.de/documents>

EN Thank you for choosing our product! Please take a few minutes time to read through the product manual. If you have any questions, please do not hesitate to contact us. You can find our contact information at the bottom of this page. Please visit our website to find the extended digital version of this manual and for further documents to our products:

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4.

EN - Instructions

1. Unfold the mask and make sure that the metal nose clip is pointing upwards before putting the mask on. You will find the rubber loops on both sides. Take them with hands and carefully pull the mask towards your face. Please make sure it covers your chin and nose.
2. Carefully pull the rubber loops over your ears. Make sure the mask fits properly and is comfortable to wear.
3. Fix the mask in place by pressing the adjustable metal nose clips onto the bridge of the nose.
4. Please make sure the mask is applied correctly by placing both hands on the mask to hold it in place. Take a deep breath in and out. The respirator should deflate and then inflate. If you notice an air leak at your nose, adjust the nasal clip. Doublecheck and readjust the masks if necessary.

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