SINO EURO LEAD

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<https://www.sino-euro.de/documents>

EN Thank you for choosing our product! Please take a few minutes time to read through the product manual. If you have any questions, please do not hesitate to contact us. You can find our contact information at the bottom of this page. Please visit our website to find the extended digital version of this manual and for further documents to our products:

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4.

EN - Instructions

1. Turn the mask with the white inner side towards you and make sure that the metal nose clip is facing upwards before putting on the mask. You will find the rubber loops on both sides. Take them with both hands and carefully pull the mask towards you. Please make sure the nose is covered first.
2. Gently pull the rubber loops over your ears. Make sure the mask fits properly and comfortably and covers your nose.
3. Hold the mask in place at your nose with one hand while carefully pulling the bottom of the mask over your chin with the other hand. Then fix the mask in place by pressing the adjustable metal nose clip onto the bridge of the nose.
4. Please make sure the mask is sitting correctly by placing both hands on the mask so that it does not slip. Take a deep breath in and out. The mask should contract slightly and then inflate slightly. If you notice an air leak at your nose, adjust the nasal clip. Doublecheck and readjust the mask if necessary.

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